

OPTRON PRO: Quick Setup Guide

Contents:



Step 1: Attach the clamp



Step 2: Insert the System Extension (optional)



Step 3: Carefully slide the swivel arm into the notch



Step 4: Connect the A/C cable to the power supply



Step 5: Connect the power supply cable to the OPTRON



Step 6: Setup the monitor (power cable, VGA cable)



Step 7: Connect the monitor cable to the monitor port on the OPTRON




Step 8: Make sure that the PC/OPTRON Toggle is pushed in



Step 9: Plug power supply and monitor cable into a wall socket.



Step 10: Turn on the OPTRON and the monitor.

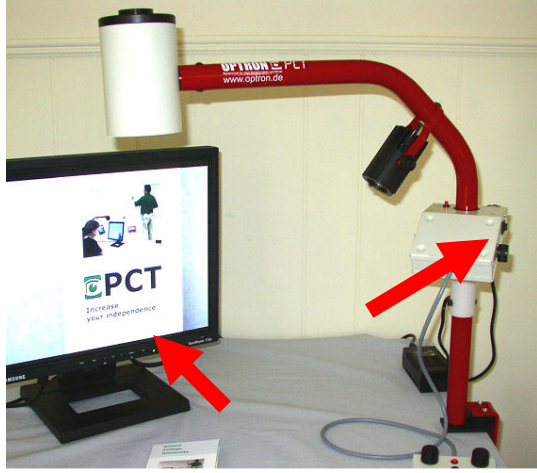

Camera Positioning:



Turn the black camera ring on the white base for an upright and level picture.



Turn the camera head to switch from distance to desktop viewing. (Desktop viewing: Close-up lens required).



Turn the swivel arm to position the camera at the ideal viewing angle.

Close-up Lenses:



Lower height: Close-up lens without dots.

Higher height: Close-up lens with dots.



Functions:


PC/
OPTRON
Toggle

Brightness

Zoom out

Monochrome/
Color
Selection
Knob

Spotlight on/off

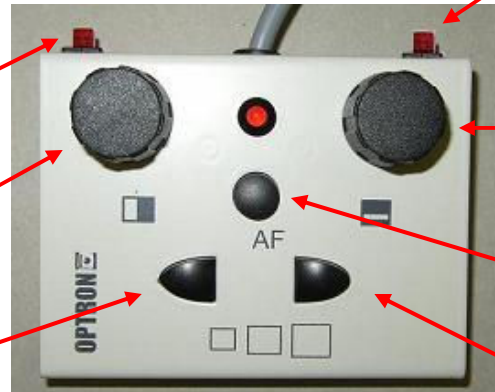

Reverse
Polarity
Toggle

Reading
Line

Auto Focus
on/off

Zoom in



Power
Switch

