

OPTRON MOBILE: Quick Setup Guide

Contents:



Step 1: Attach the clamp



Step 2: Insert the System Extension (optional)



Step 3: Carefully slide the swivel arm into the notch



Step 4: Connect the A/C cable to the power supply



Step 5: Connect the power supply cable to the OPTRON



Step 6: Setup the monitor (power cable, VGA cable)



Step 7: Connect the monitor cable to the monitor port on the OPTRON




Step 9: Plug power supply and monitor cable into a wall socket.



OPTRON ASSISTIVE TECHNOLOGIES
1-888-567-8766

OPTRON 
Look at it.

Step 10: Turn on the OPTRON and the monitor.

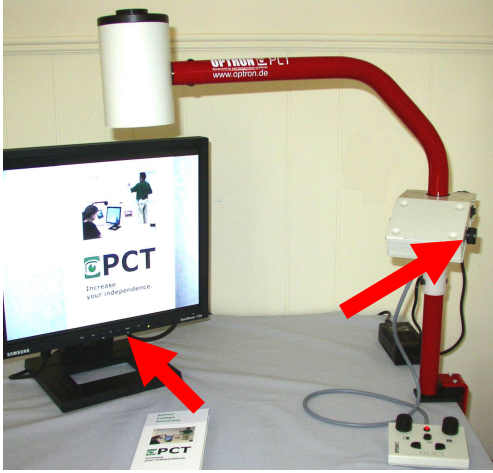

Camera Positioning:



Turn the black camera ring on the white base for an upright and level picture.



Turn the camera head to switch from distance to desktop viewing. (Desktop viewing: Close-up lens required).



Turn the swivel arm to position the camera at the ideal viewing angle.

Close-up Lenses:



Lower height: Close-up lens without dots.

Higher height: Close-up lens with dots.



Functions:

