



OPTRON *mobile*: Quick Setup Guide

Contents:



Step 1: Attach the clamp



Step 2: Insert the System Extension (optional)



Step 3: Carefully slide the swivel arm into the notch



To gain a maximum of desk-top workspace, you can attach the Main Box underneath the table top by using the attached Velcro® strips. **Please be aware that these strips are industrial grade and might be hard to peel off and/or might leave some residue on your table or desk.**


Step 5 (optional): Attach the Main Box underneath or on the side of your table by using the attached Velcro® strips.



Step 7: Setup your monitor according to the instructions.



Step 6: Connect the power supply, the control panel, the camera and the monitor to the main box.

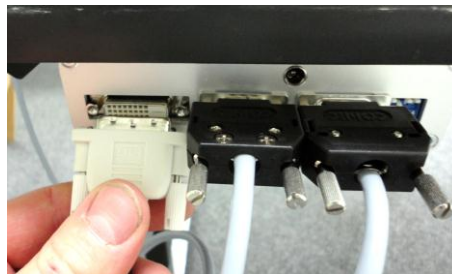

Step 9: Connect the OPTRON *mobile* power supply and the monitor cable to an outlet.



Step 10: Turn on the OPTRON mobile and the monitor.




Camera Positioning:



Turn the black camera ring on the white base for an upright and level picture.



Turn the camera head to switch from distance to desktop viewing. (Desktop viewing: Close-up lens required).

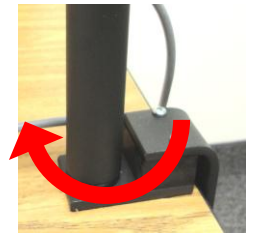

Turn the swivel arm to position the camera at the ideal viewing angle.

Close-up Lenses:



Lower height: Close-up lens without dots.

Higher height: Close-up lens with dots.




Control Panel Functions:

Auto Focus on/off

Zoom in

Image Mode

Power on/off

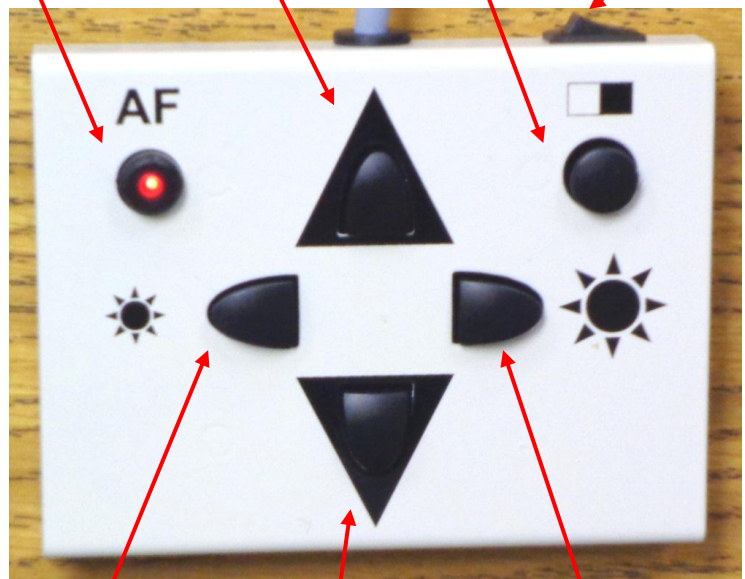

Darker

Zoom out

Brighter