

# **mv** OPTelec Clearview<sup>+</sup>: Quick Setup

Please make sure that you read and understand the Optelec Clearview+ user guide in its entirety. For questions please call Optelec (1-800-826-4200) or Magnified Vision (1-888-567-8766).

## Contents:



**Step 1:** Place base on table as shown




**Step 2:** Remove cover on top of camera unit



**Step 3:** Carefully insert monitor arm into receptacle



**Step 4:** Carefully slide monitor onto the monitor arm

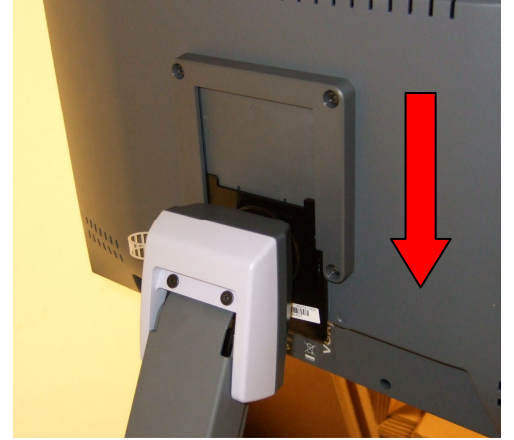


**Step 5:** Rotate screen 90 degrees



**Step 6:** Connect monitor cable to the screen. Tighten thumbscrews




**Step 7:** Plug power cord into monitor

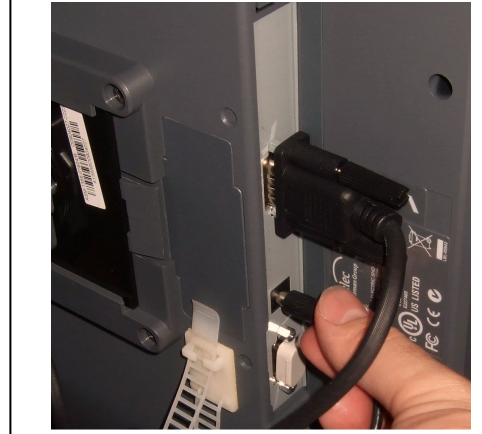

**Step 8:** Secure cords in the loop of the plastic tie



**Step 9:** Plug monitor power cord into base

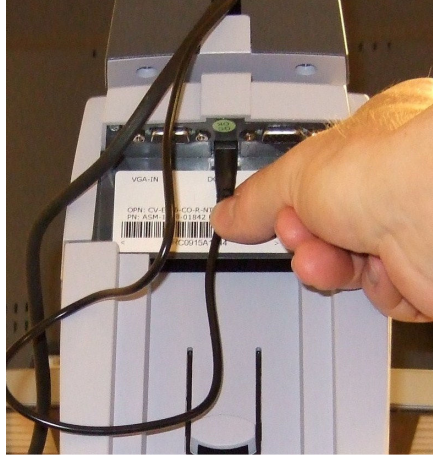

**Step 10:** Connect monitor cable to the base. Tighten thumbscrews



**Step 11:** Connect cord from power supply with the flat side of the plug facing away from the unit



**Step 12:** Slide cover on back of the unit. Make sure that cords are not hampered




**Step 13:** Connect power cord to power supply



**Step 14:** Turn unit into correct position. **Lift unit only by handles on sides underneath reading tray**



**Step 15:** Plug power cord into wall socket

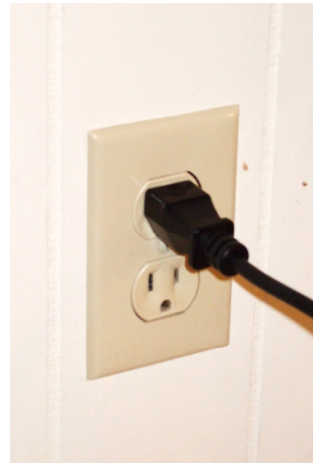

**Step 16:** After red light on control button is off, turn on the unit

